AiLink

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Dear customers,
Thank you for your purchase and interest into our scale. Use this manual together with the maual for the scale.This manual is focused only on the application and its options. It is not focused on the usage of the scale.

Illustrations and translations of texts or of activating elements applied in the application may slightly differ from the illustrations and descriptions specified in this manual. Therefore, don't get scared if there are differences in your application compared with the illustrations in the manual. The illustrations provided may also cover different measurements, so don't be afraid if you see a discrepancy between the results.

| | |
|---|-----------|
| CONTENT | |
| BASIC STRUCTURE OF APP'S MAIN MENU | 20 |
| BASIC STRUCTURE OF HOME SCREEN | 21 |
| USER MANUAL FOR THE AILINK APP | 22 |
| 1. APPLICATION INSTALLATION | 22 |
| 2. ACCOUNT REGISTRATION AND LOGIN | 22 |
| 3. CONNECTING THE DEVICE | 22 |
| 4. BASIC ADVICE AND TIPS FOR USE | 24 |
| 5. MEASUREMENT | 24 |
| 6. RESULT VIEW | 25 |
| 7. SHARING RESULTS | 25 |
| 8. FRIEND – RESULT SHARING | 26 |
| 9A. SETTINGS - GENERAL | 26 |
| 9B. SETTINGS - APPLICATION | 26 |

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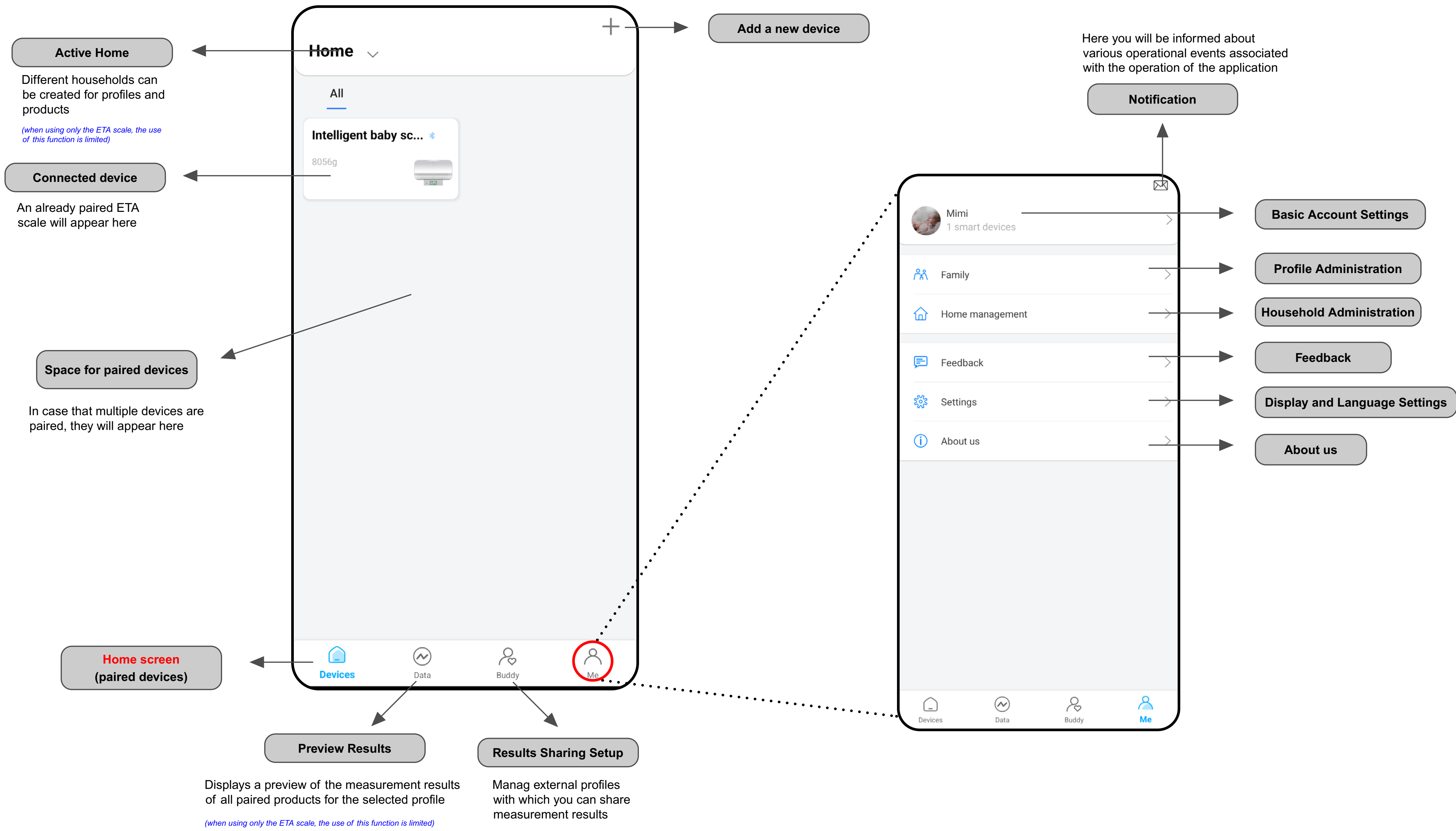
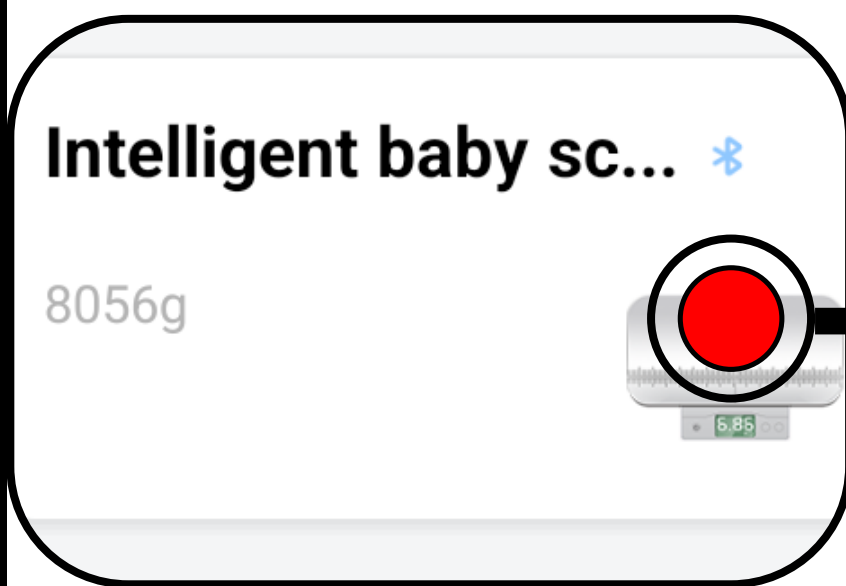
Information Bar

Profile Administration

Allows you to switch between profiles, including the ability to create new profiles

General settings and sharing results

Device name, change units, delete device and share menu - allows you to share the current measurement result via supported social networks (WeChat, Facebook) or by saving the recording to the device storage

Measured Weight

Zero (TARE)

Allows you to reset results and reweigh (or reset pad weight, etc.)

Create a new measurement record

Age of child (M<months>-D<days>)

Result section

Brief commentary on the values

Difference from last measurement

USER MANUAL FOR THE AILINK APP

Dear customer,

Thank you for your trust in ETA product. This manual is intended for all owners of the ETA7306 scale who want to reach its full potential with the AILINK application.

In case of questions about the function of the application, or in case of any problems, first go to the above-mentioned website and even if the information provided there does not help you to solve your request, you can contact us at the information e-mail: info@etasince1943.com

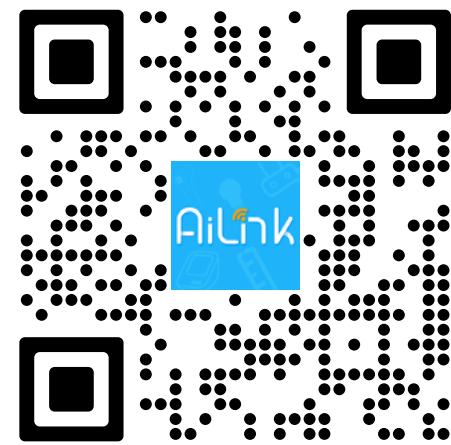
Further information, advice and tips can be found also on our websites:

www.etasince1943.com/care

We are constantly updating this website to gather the latest information about the product.

1. APPLICATION INSTALLATION

You can find this application in the database of Apple Store or Google Play, or can also use this QR code for quickening:

Download and install the application to your mobile phone (tablet).

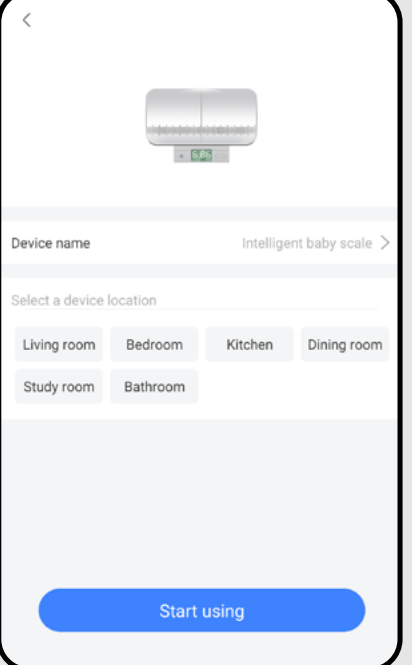
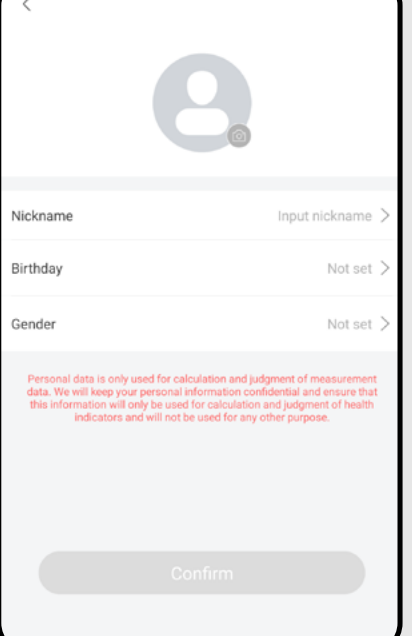
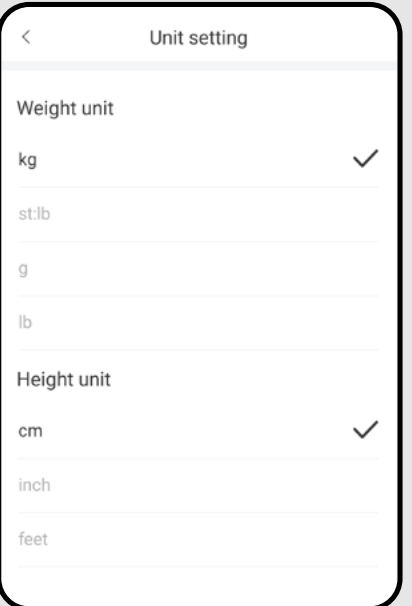
| KOMPATIBILITA | |
|--|---------------------------------------|
| Mobile phone's (tablet's) operation system | Android 8.0 and up IOS 11.0 and up |




2. ACCOUNT REGISTRATION AND LOGIN

| | | |
|---|--|--|
| 1 | | <ul style="list-style-type: none"> Once you have downloaded the application, turn it on. The basic login screen appears. The screen also allows you to switch to register a new account, reset your password if it is lost, or log in using third-party applications. If you do not have an account, proceed to „Register“ (or sign in with a third-party application, if applicable) Enter your account name to log in (email) and password, then click on „Log in“. In the case of the Android OS, you must check the terms of use box in the lower left corner before logging in. |
| 2 | | <ul style="list-style-type: none"> Enter a valid e-mail address and choose a secure password. Confirm your acceptance of the services and policies and confirm the process by clicking „Register“. |
| 3 | | <ul style="list-style-type: none"> If you've lost your password, you can renew it by means of the e-mail address used for creating the account. Enter this e-mail and as soon as you receive a verifying code, write it to the respective field. Remark: It could take a certain time until the verifying code comes up. Please be patient. At the same time, check that the code is not blocked by the spam basket/filter of your e-mail client. Ideally, use common email addresses / clients (eg. Gmail, Yahoo, etc.) |

3. CONNECTING THE DEVICE

| | | |
|---|--|--|
| 1 | | <ul style="list-style-type: none"> After successful login, the home screen will appear, in this case without a paired device. Click to connect the application with the scale „Add device“ <p><i>* If you already have a device paired, use the „+“ icon in the upper right corner, which will allow you to add another device in the same way.</i></p> |
| 2 | | <ul style="list-style-type: none"> A list of different types of devices for which the application is intended appears. <p>Note: ETA uses the application only for the ETA7306 and ETA8781 scale. No other devices listed in the application are currently supported.</p> |
| 3 | | <ul style="list-style-type: none"> The best way is to automatically find the device you go to by clicking „Devices nearby“. The search for nearby devices starts automatically. Press the switch button to switch the scale on. When the scale appears in the application, click on its icon. |

| | | |
|---|---|--|
| 4 |  | <ul style="list-style-type: none"> The pairing process will start. |
| 5 |  | <ul style="list-style-type: none"> In the next step, the application offers the possibility to add your own balance name and assign and categorize the balance according to the room in which it will be used. Click to confirm „Start using“. <p style="text-align: center;">Note The room categorization feature makes more sense when using multiple products that the application supports. In our case (ETA scale only) you do not have to use this function.</p> |
| 6 |  | <ul style="list-style-type: none"> If this is your first login, the app may prompt you to create a profile. In order to use the baby scale, the profile must be created for the child - that is, with the child's date of birth and gender. If the profile was kept with a birth date far in the past, the app would not work properly and would not display all options. |
| 7 |  | <ul style="list-style-type: none"> In the last step of the device setup, you will be offered the option to set the weight and height units. Click „OK“ to confirm. <p style="text-align: center;">Note To cancel pairing / deleting the device, do so in the settings section, see 9B-2</p> |
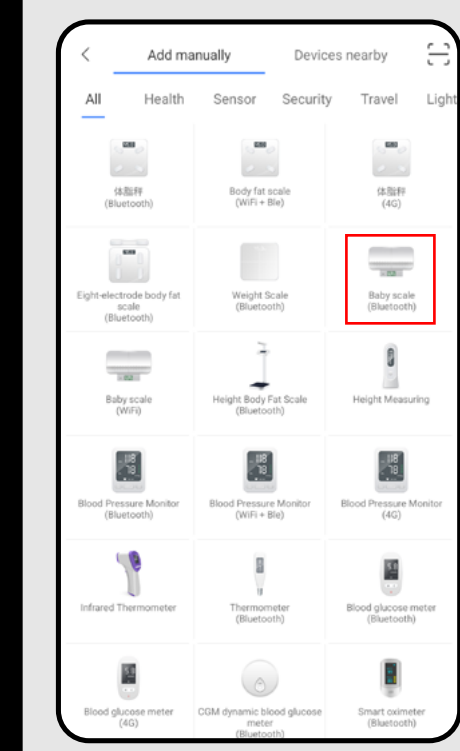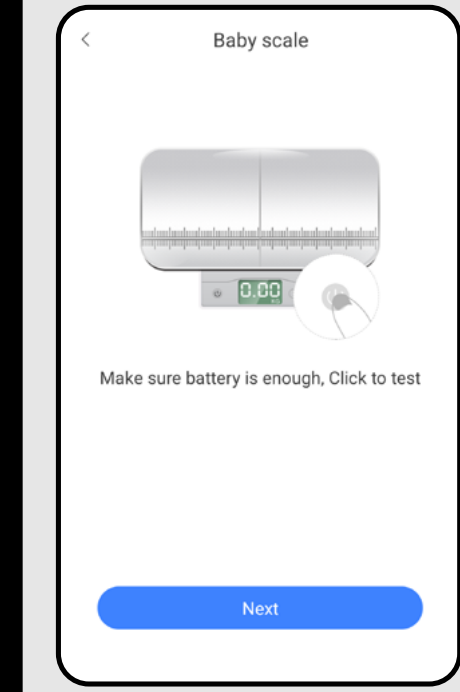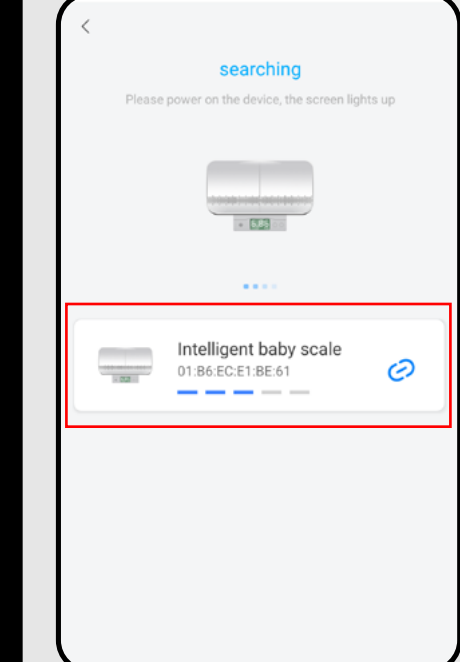
| | |
|---|--|
|  | <ul style="list-style-type: none"> The second way is manual pairing. Select "Baby scale (Bluetooth)" from the list of devices. |
|  | <ul style="list-style-type: none"> Click on "Next". |
|  | <ul style="list-style-type: none"> Turn on the scale, it will automatically send a signal that your mobile phone will pick up and displays the device it's looking for in the app. Confirm by clicking on the searched device. The next steps are identical to the automatic pairing (from step 3.5). |

4. BASIC ADVICE AND TIPS FOR USE

Using the ETA scale with AiLink is intuitive and we did not notice any significant problems or difficulties during our testing you might encounter while using it. Nevertheless, smart products generally have an increased risk associated with incompatibility of mobile devices or problems with wireless connectivity. Therefore, we prepared a set of basic advice and recommendations on how to solve problems characteristic of the smart products segment.

1. Smart device (mobile phone, eventually tablet):

- a) always use the current version of the application downloaded from official sources
- b) make sure your devices have supported versions of Android and iOS.
- c) have the Bluetooth function active
- d) have an active internet connection for proper communication of the application with the cloud server
- e) allow the application the required permissions, which in some cases depend on the type of operating phone.
In case of problems, make sure that you do not have any parent system settings within your phone, for example, that prevent applications from using certain permissions - in which case you must grant the application an exception.
- f) the communication between the balance and the phone may be disturbed by some programs (eg VPN, firewall, etc.) that you use in the phone.
In case of problems with pairing or normal operation, turn off these programs.
- g) In case of problems, try pairing with another phone to avoid the problem in that phone.

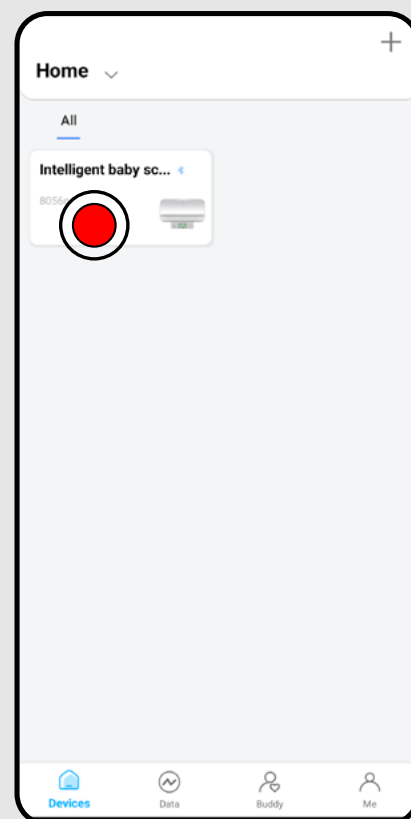
2. Scale

Always make sure you use sufficiently charged batteries. Weak batteries can negatively affect the function of the Bluetooth connection or the measurement process itself. In some cases, the batteries that came with the product may also have reduced capacity due to storage and shipping, so it is a good idea to try new batteries if you have any problems.

- Try creating a new account to avoid the problem of incorrect account activation;
- Keep your smart device close to the scale (preferably within a distance of approx. 2 m) during pairing and use.


5. MEASUREMENT

1


- After switching on the application and successful connecting, click on the field / icon of the paired balance.
- This will take you from the general part of the application to the section for controlling and managing the scale itself.

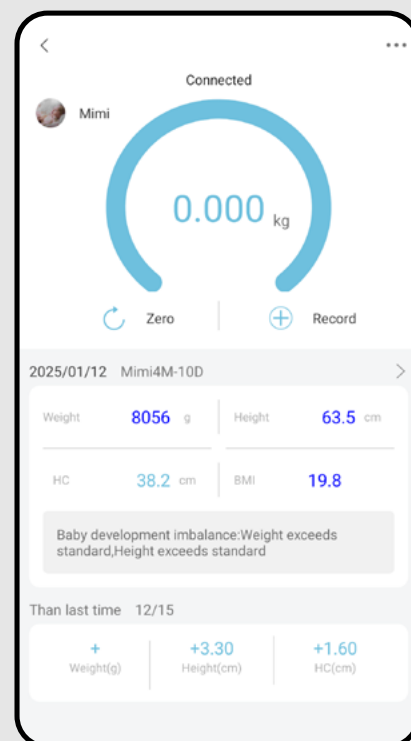2


- The basic measurement screen is displayed.
- Click on the profile icon in the upper left corner to select a profile (if you have more than one).
- Turn on the scale and wait for the scale to connect to the app - the information bar will display the message **"Connected"**.
- When the scale is switched on, the device will automatically connect. If the information bar says **"Connection failed..."**, just click on the notification and the connection will be restored. However, the scale must be switched on.
- Before the actual weighing, measure the child's current height and head circumference - these values can (optionally) be added to the weight value.

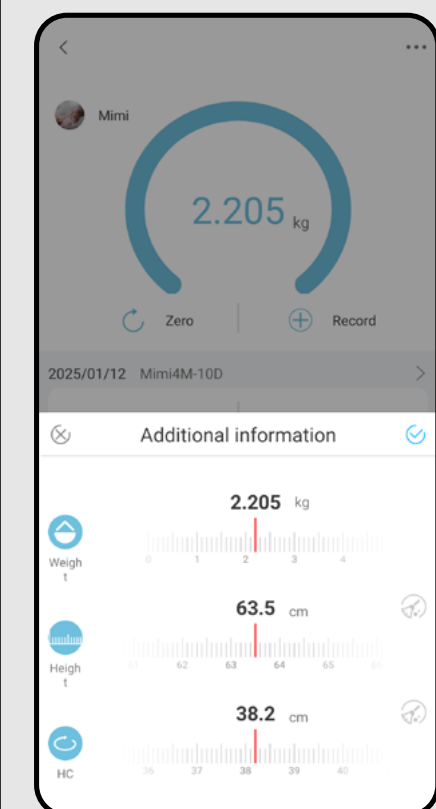3

- Now you can start the practical weighing - follow the operating instructions that came with the scale.
- Once weighed, the data will be automatically transferred to the app and a screen will appear with the option to add the height and circumference of the child's head. Find out these parameters before the actual weighing and add them in this step.
- If you do not want to enter the height or girth, click the "🚫" icon to the right of the parameter.
- To create a record, click on the "📌"

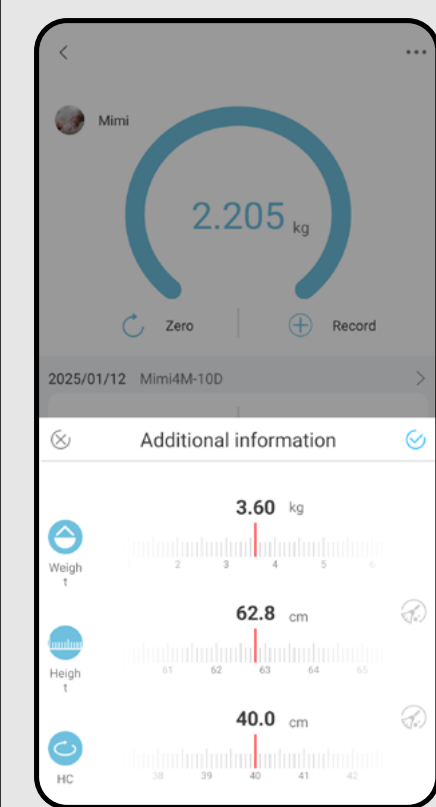4


- The measurement results can also be entered manually. If you want to create a record of the measurement without the actual weighing, press **"Record"** on the main screen and enter the values manually, including the date, to which the values relate.
- In this way, values can also be added retrospectively.

NOTES

- If the balance connection has failed, make sure you have Bluetooth activated on your phone, then click on the text in the information bar to reconnect. Alternatively, read the instructions in Chapter 3.
- If the measurement or analysis failed, try the whole weighing / measuring process again. Alternatively, read the instructions in Chapter 4.
- Once a record is created, the tracked parameters cannot be edited/ added back. However, a new record can be manually inserted to replace an incomplete measurement (including selecting a backdate).
- If the child is constantly moving and the value does not stabilize, try pressing **"Record"** in the app (at the point when movement is lowest) to force the measurement result to stabilize.
- The record contains a brief verbal assessment of the child's development according to the WHO growth standards. These assessments and standards are indicative and no medical conclusions can be drawn from them.


6. RESULT VIEW

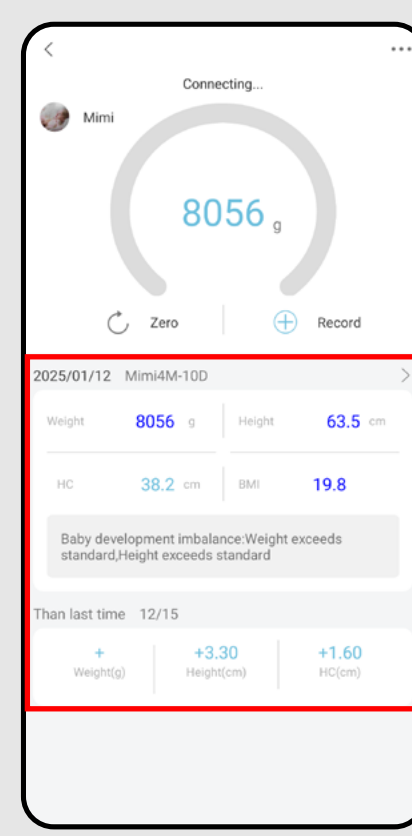1


- Click on one of the entries on the main screen. This will take you to the section for recording measurement records and graphical representation of measurement results.
- You can scroll through the individual records.
- The records also contain a brief text commentary on the child's development.

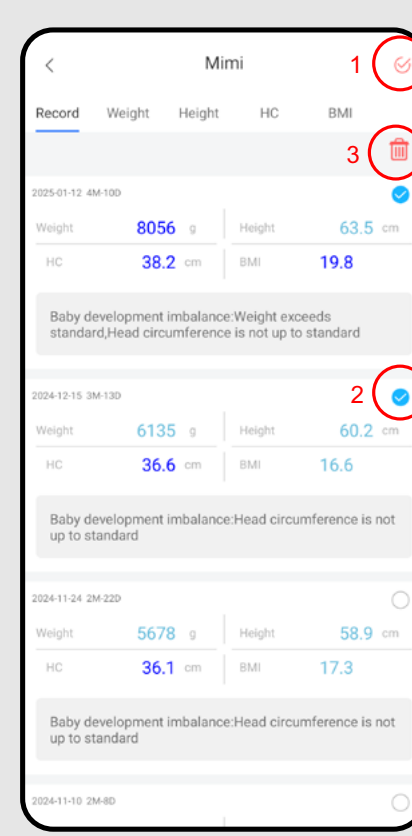2


- To delete a record, click on the pencil icon in the top right corner (1).
- Then click to select the record or multiple records (2) you want to delete.
- Finally, confirm the selection by clicking on the trash can icon in the upper right corner (3).
- Finally, confirm the process by clicking "OK".

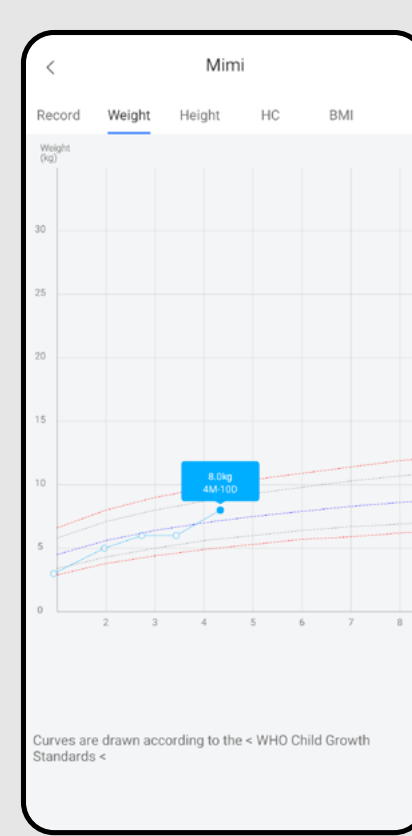3


- Click on the individual parameters of the monitored measurements in the top bar to go to their graphical representation.
- The WHO growth curves are also shown in the graph - so you can see the approximate development within these standards.
- First, the "Weight" is available.

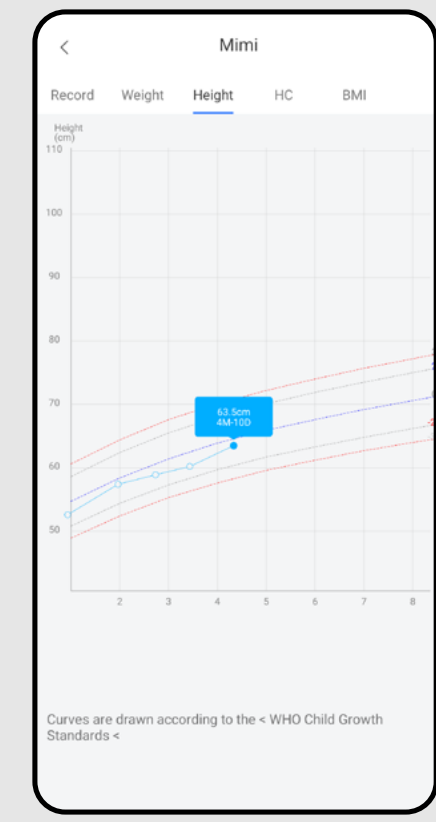4


- Another monitored parameter is "Height".

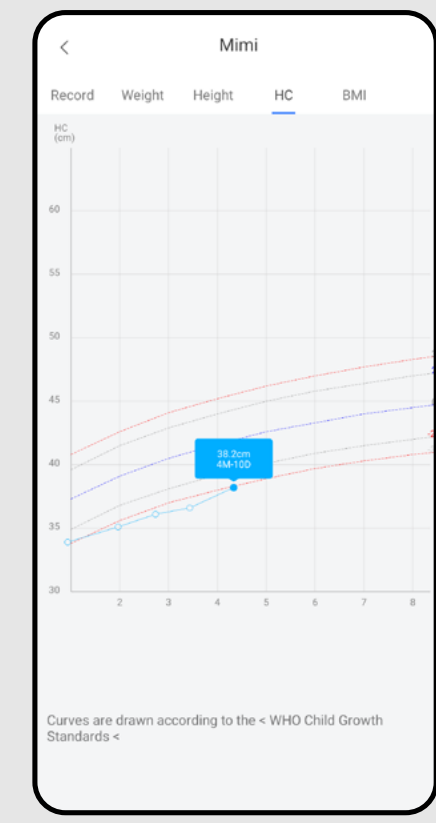5

- Another monitored parameter is "Head circumference (HC)".

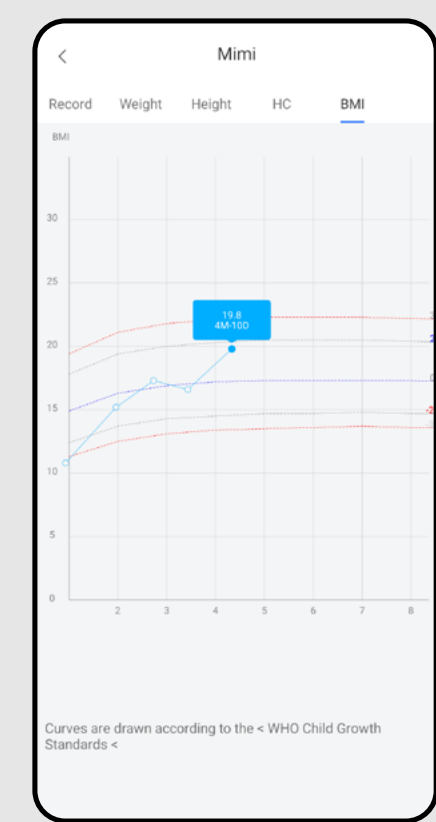
6

- Another monitored parameter is "BMI".


7. SHARING RESULTS

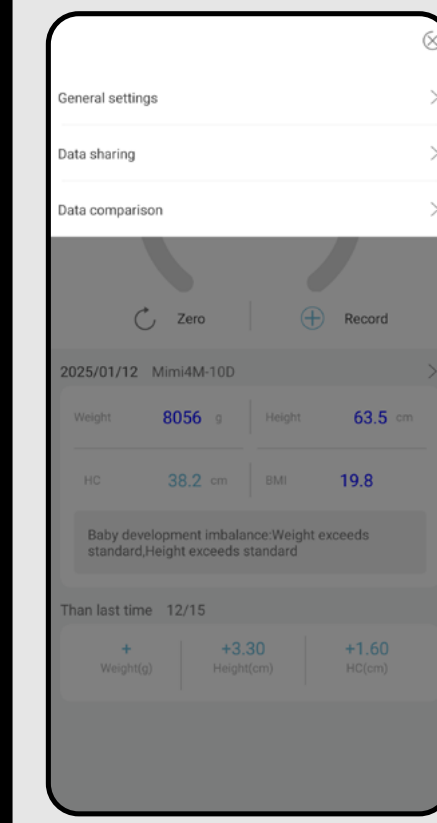1


- On the main screen, click on the "... " icon in the top right corner.
- The settings menu will appear, where you have 2 basic options to view and share your results:
A) **Data Sharing**
B) **Data Comparison**

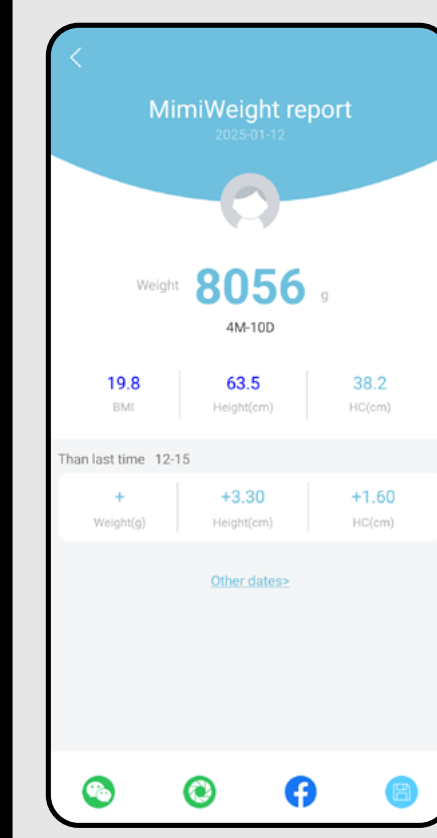2

- **Data Sharing**
- This is a simple record of a specific measurement. Click on "More Dates" to go to a list of measurements sorted by date. Click on a specific measurement to view the report.
- The report can be further shared using the Facebook app, or by clicking on the floppy disk icon to download it to the device storage where it can be further work with the image in various other applications.
- Note
- The green icons belong to the WeChat app, which is not recommended for use in Europe.

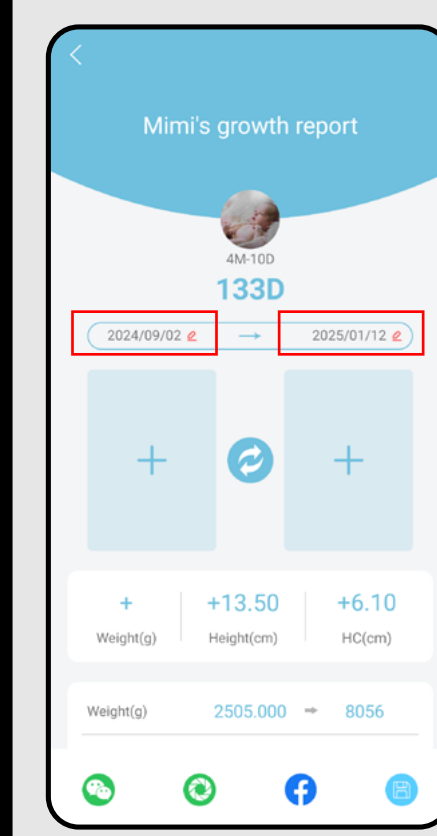
3

- **Data comparison**
- This is a simple record of a specific measurement. Click on the measurement date icons to go to a list of measurements sorted by date. Click on a specific measurement to select it. Do this for the date on the left and right side.
- Select an older measurement on the left and a more recent one on the right. The report will create a comparison of the two measurements and show the differences.
- Photos can be added below the date fields from the period to give a better visual contrast of the change.
- The method of sharing is the same as the point above.


8. FRIEND – RESULT SHARING

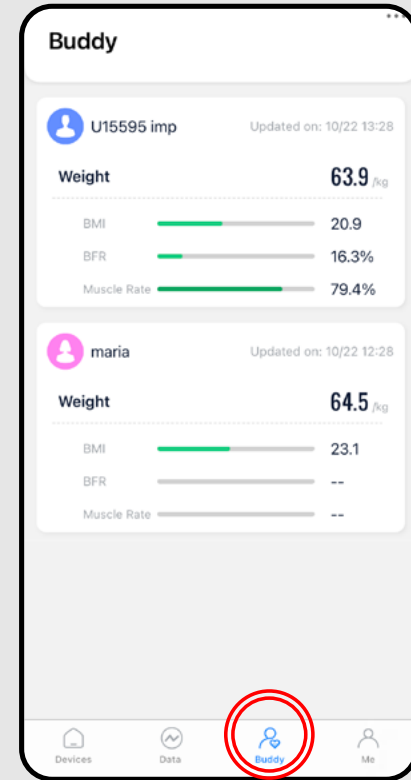1

- In addition to the ability to share your own results, the application also allows you to link AiLink user accounts to each other.
- This way you can view the results of the person you are connected to and vice versa - that person will see your results.
- Example: in the illustration on the left you can see that the current profile shares its results with the user „imp“. IMP has multiple profiles, but has chosen to share only his primary profile and profile of „Maria“.

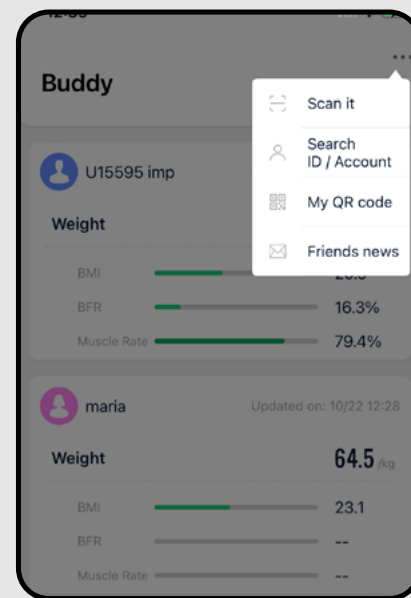2


- To add a new friend, one of the accounts must enter the „Buddy“ and press „...“. Then you have 2 options:
 - 1) You can use the „Scan it“ option Scan the QR code you receive from the other account you want to share data with from your phone. You can get the QR code by clicking on „My QR code“, on the phone of the other account, scan this QR code and confirm the sharing of profiles.
 - 2) Use the „Search ID / Account“ option to enter the e-mail address of the other account and confirm.
- As soon as you send a request to the other party, a notification will appear in the application in the „Friend“, section, one of the displayed red markers (👤/👤🔴). will appear next to the icon. Then select „...“, go to „Friends' News“ and confirm your request.

Note

The account that sends the request and the account that receives the request can choose which sub-profiles to share with the other party when sending / receiving the request.

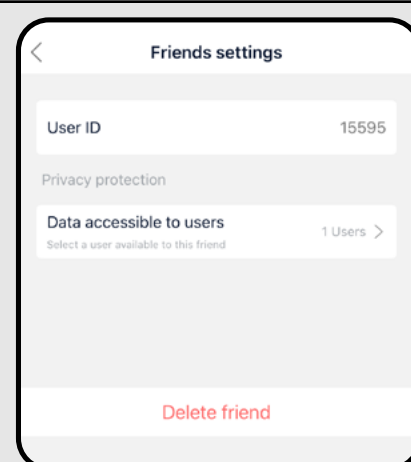3


- If you want to change some parameters of the shared account (eg account name, delete share, change shared sub-profiles), click on the profile in the „Buddy“ section and then on the settings icon „⚙️“.

9A. SETTINGS - GENERAL


The application allows you 2 basic setting options. Settings within the general part of the application (which is common for the management of all products used in the application) and settings within the application part intended for the ETA scale. In both cases, some settings overlap, some are specific.

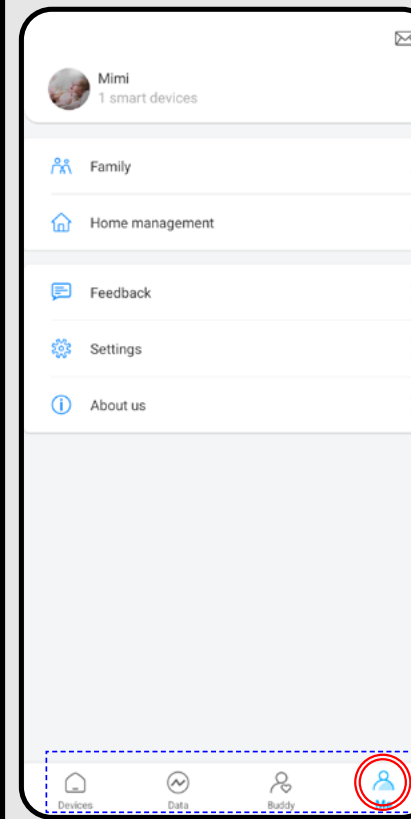1


- **General section allows you to set:**
 - a) **Main profile adjustment** (see below 9A-2)
 - b) **Profile management** (see below 9A-3)
 - c) **Home management** - Management of households, rooms and related products. This function is unnecessary for the purpose of using only the ETA scale.
 - d) **Feedback** - It is possible to give a feedback to the developers of the app. Please, do not use this option. We recommend to use the form in the app. (See below 9B-2)!
 - e) **Settings** - Ability to switch the application language and display connected devices.
 - f) **About us** - Displays the version of the application and the option to view the terms and conditions of the application.

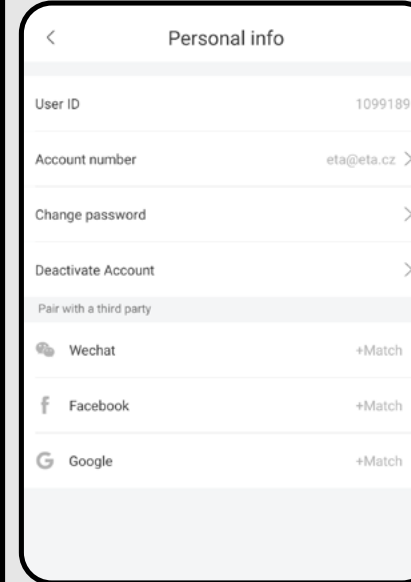2

- Click on the profile icon (👤) to go to the profile settings.
- Here you can find the account ID, change the password, or deactivate the account.
- The options of third-party applications (WeChat, Facebook) in this part of the application are only used to inform you whether you are logged in to your account using these services.

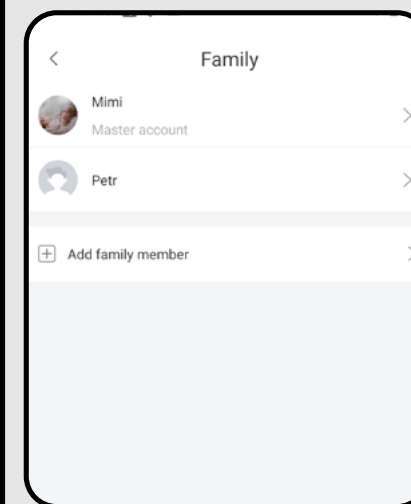
3

- Section „Family“ allows easy profile management. In the menu you can see both the main profile and all created sub-profiles.
- After clicking on the profile, you can change the basic information (photo, nickname, birthday, gender) or you can delete the profile.
- To create a new subprofile, click on „Add family member“.


9B. SETTINGS - APPLICATION

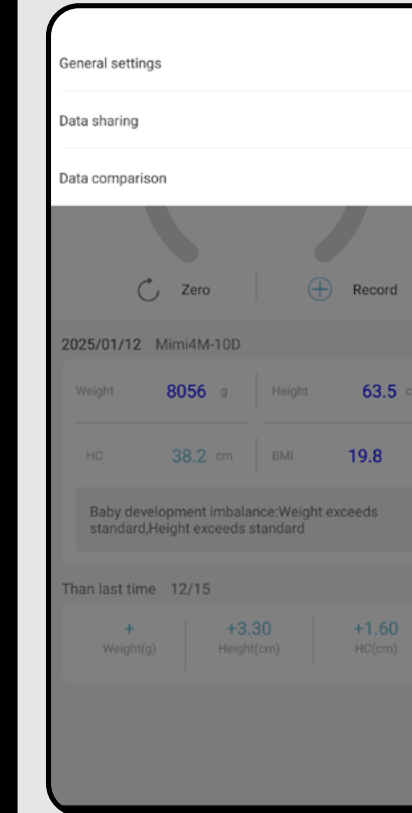1


- On the main screen, click on the „...“ icon. in the top right corner.
- Click on „General settings“.

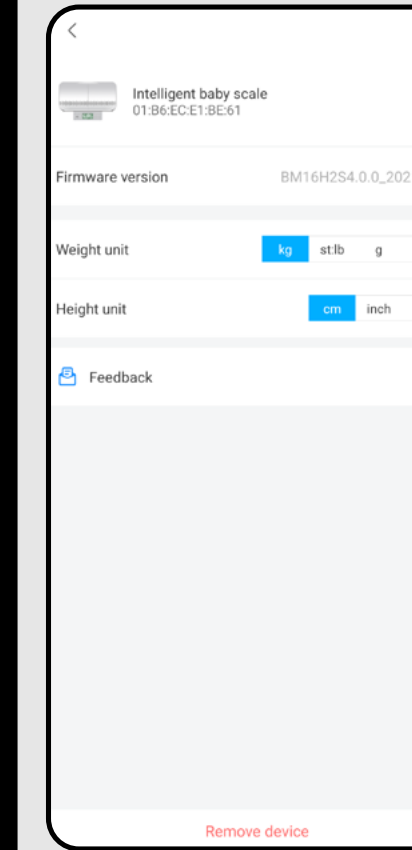
2

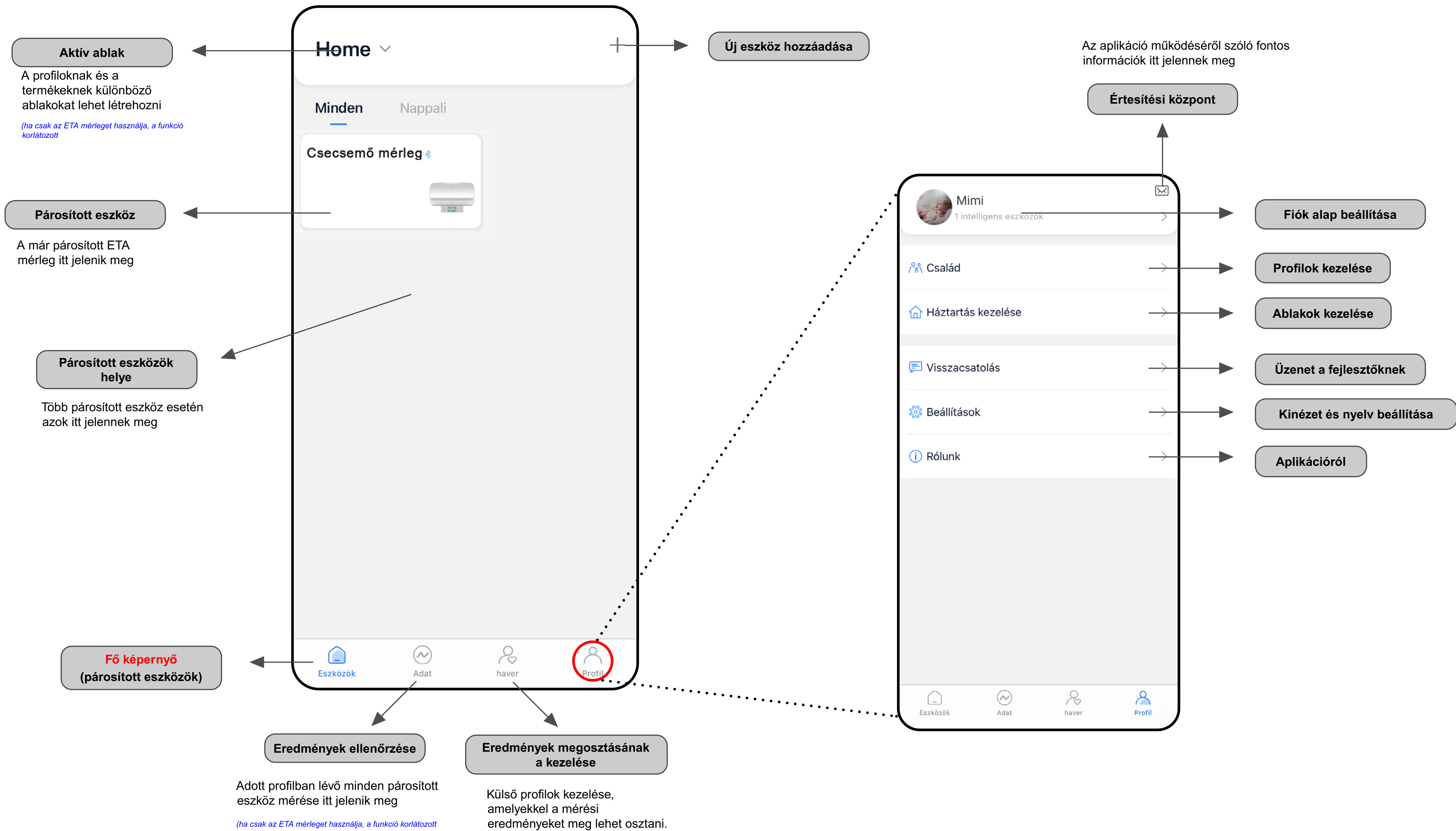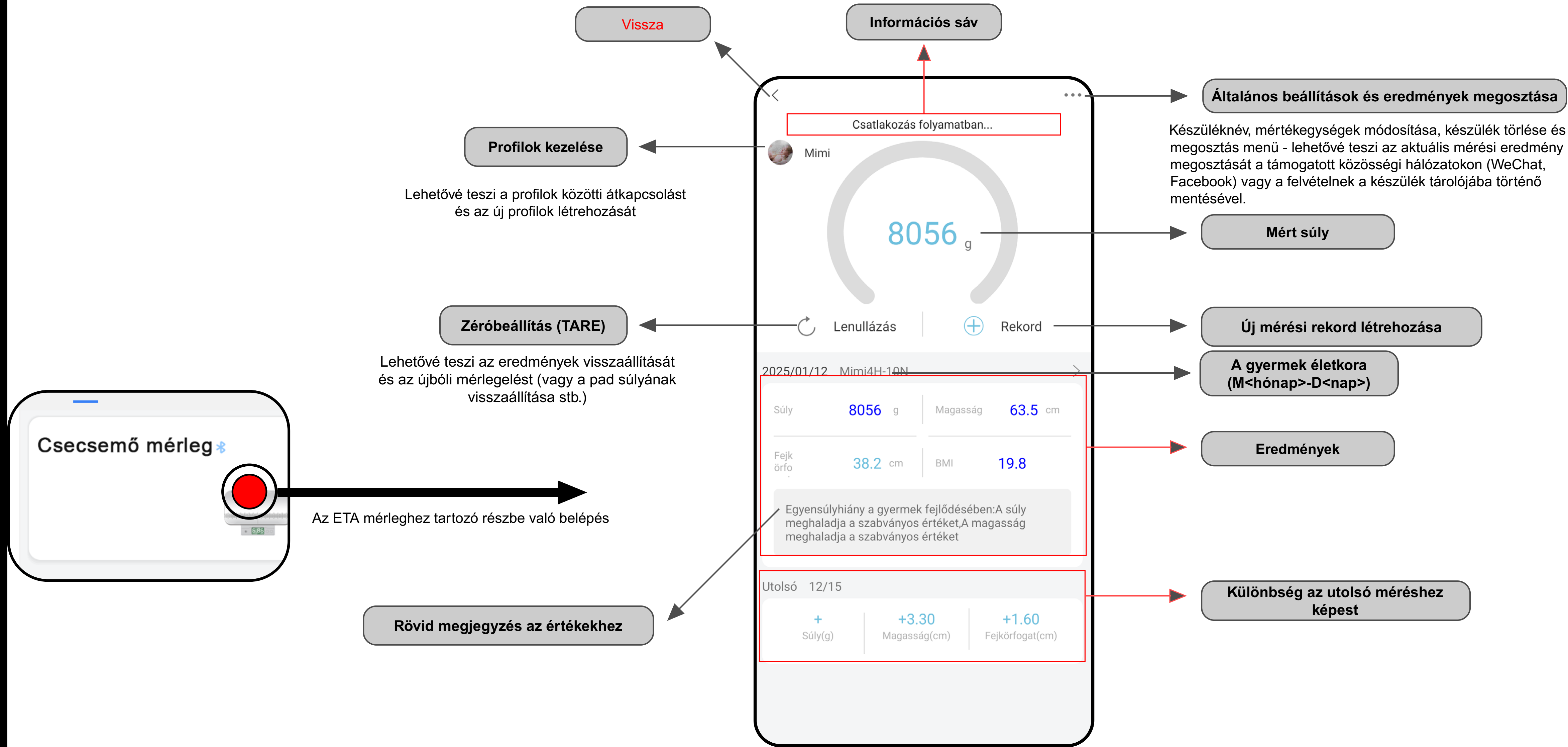- **In the application section you can set the units** of weight and height. Check the firmware version, rename the weight and send feedback to the developer. **Make best use of this option** (rather than the field in the general settings section).
- Then at the bottom of the screen you will find the „Remove device“ option, which will delete the device (unpair it).

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